

Modern cosmetic toe nail correction

Our feet carry us around all of our lives, day in, day out, patiently and dutifully. They have to tolerate every day stress, like standing too long at work, tight shoes, insufficient airing and too much moisture. Our feet deserve better.

Our feet want to be seen, well cared for and beautiful. Today, nail diseases can be treated permanently and successfully with nail cosmetic treatments. This has become possible by the development of modern corrective materials. The modern foot care product WILDE-PEDIQUE plus is especially effective for the treatment in the studio of your individual problem.

What does modern toe nail correction mean?

WILDE-PEDIQUE plus is a means to restore the toe nail partially or completely thus providing an attractive cosmetic result. The 1-phase resin is a light-curing synthetic resin which is cured in a light unit. WILDE-PEDIQUE plus is an elastic gel which adjusts to the movements of the nails and feet. Thus the artificial nail feels completely natural, since it fits perfectly to the natural nail. Your WILDE-PEDIQUE plus specialist will determine the individual shade of the resin according to disease and problem. Thus discoloring of the nail can be covered or the nail bed color can be improved.

WILDE-PEDIQUE plus Step by Step Application



1 The nail before the treatment.



1 One layer of WILDE-PEDIQUE plus is applied onto the natural nail.



2 The nail is now modeled with WILDE-PEDIQUE plus and the shape determined. The material is cured in a special light unit for two minutes. The light is not dangerous and doesn't cause any changes of the skin.




2 At the free end of the nail the light-curing French polish is applied and then cured in a special light unit for two minutes.



3 Your nail is finished with a subtle surface shine.



3 Your nail is finished and has a permanent French look.

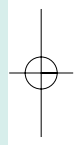

Your WILDE-PEDIQUE plus specialist also offers several sealing alternatives. Ask for the product SEALANT if you desire a high gloss finish. Several colored sealants are also available.

Don't hesitate to ask in your WILDE-PEDIQUE plus specialist.

WILDE-PEDIQUE plus combined with nail braces

If you suffer from ingrown nails your specialist might recommend the use of a brace. WILDE-PEDIQUE plus ensures optimal sealing of braces. The surface is even, can be polished and does not have any bumps, which could damage nylons.

WILDE-PEDIQUE plus used for subsequent treatments

If there are no problems (pressure points, lifting or special diseases) the synthetic material grows out with the natural nail. Theoretically WILDE-PEDIQUE plus grows out with the natural nail within 10 months. In order to ensure a permanent and attractive result, WILDE-PEDIQUE plus should be refilled. Consult your specialist when the time is right for a follow-up appointment.

Permanent French look

By using modern synthetic resins a permanent French look is possible without a long drying period. WILDE-PEDIQUE plus is then applied in two colors, rosé and white. The free nail end is white and the nail bed softly rose-colored.

There is a video available explaining these techniques. Ask the specialists in the studio.

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
Preventive foot care is the best way to avoid foot problems. Your studio will be happy to arrange an individual care program for your feet.

Your WILDE Team wishes you strong and healthy feet



WILDE COSMETICS GmbH
Post Box 12 20 • 65368 Oestrich-Winkel
Tel.: +49 (0) 67 23 60 20-0 • Fax: +49 (0) 67 23 60 20-15
www.wilde-cosmetics.com • E-Mail: info@wilde-cosmetics.com
A member of WildeGroup

Your WILDE-PEDIQUE plus specialist is:



Art.: 61213-1; 04/2007



WILDE-PEDIQUE plus
Modern toe nail correction